



## Easy-Shake Dispenser

Easy-Shake is a revolutionary hand held dispenser. It allows dispensing **one** applicator at a time to the **last** applicator without touching other applicators.

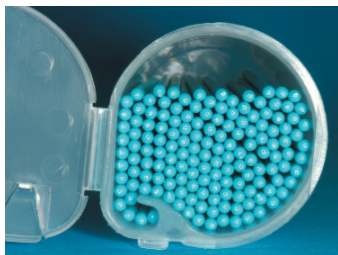

It has a semi-cylindrical shape with a flat side on one side and a round shape on the rest of the body of the dispenser. The flat side facilitates guiding the applicators to a passageway. The shield covering separates the main body of the dispenser from the passageway. The purpose of separating the main body from the passageway is to disallow applicators to pile on top of each other in the passageway. As a result only one applicator is dispensed at a time – to the last applicator. Only one applicator is touched at a time as only one applicator is dispensed. The problem of cross contamination is solved with Easy-Shake Dispenser.

### Holding Position of Easy-Shake

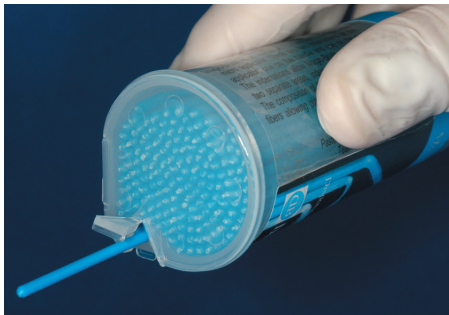


Maxi-Brush is a Trademark of Denbur, Inc.  
Patented   
Manufactured by Denbur, Inc. ©6-07  
Made in USA

### Denbur, Inc.

P.O. Box 3473, Oak Brook, Illinois 60522-3473 USA  
630-986-9667 • 800-992-1399 • fax: 630-986-9688  
E-mail: [denbur@denbur.com](mailto:denbur@denbur.com)  
[www.denbur.com](http://www.denbur.com)

# Maxi-Brush™ for Accessing Elongated Areas

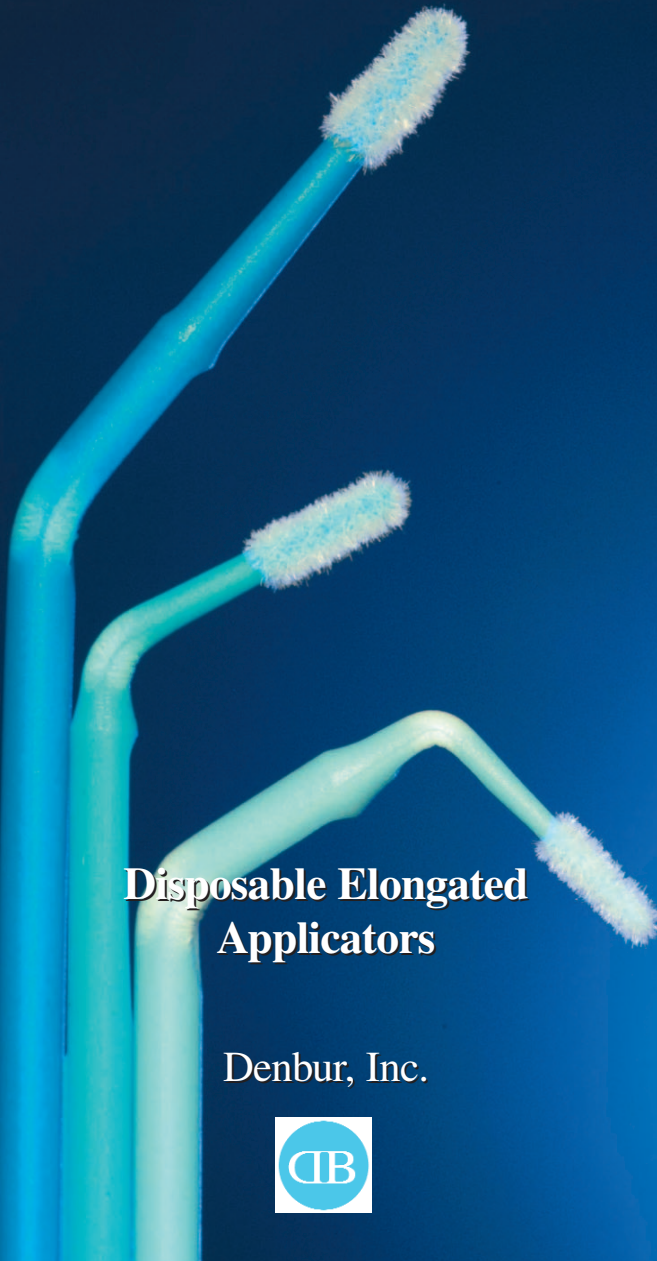

## Disposable Elongated Applicators

Denbur, Inc.





## Maxi-Brush

### Disposable Elongated Applicators

Maxi-Brush is a disposable applicator that facilitates access to narrow extended and confined spaces for a variety of applications. With its elongated and prolonged stiffer and non-lint fibers, Maxi-Brush in 3 sizes allows easy access to short, long, small, medium, and large confined spaces.

Maxi-Brush can be used to access growing canal spaces, periodontal pockets, or axial walls. It can reach and access canal spaces for post preparation, applying a bonding agent evenly and thoroughly in a post-space. It is ideal to apply uniform coats of bonding agents to an inlay preparation, or to distribute the bonding agent evenly along the entire area of an onlay preparation. With Maxi-Brush apply the bonding agent to axial walls and pulpal floor of inlay or onlay preparations in continuous motion and even amounts. For porcelain veneer preparations use Maxi-Brush to apply a bonding agent using a wand-like motion. With Maxi-Brush apply bonding agent to a traditional class I preparation, or in class II preparation use Maxi-Brush by treating the entire cavity walls in a single application.

### Sizes, Colors, and Bending Positions



- Size Small, tip @ 0.90 mm >, color Pastel Blue. The exclusive double bending feature is ideal for application from the lingual.



- Size Medium, tip @ 1.50 mm >, color Teal. The applicator in short bending position.



- Size Large, tip @ 1.60 mm >, color Process Blue. The applicator in long bending position.

All 3 sizes of Maxi-Brush can be used for various applications depending on the size of the preparation and the nature of the treatment.


- Apply bonding agent in a post-space preparation with size small, medium or large, depending on the size of the post preparation.
- Apply uniform coats of bonding agent to an inlay cementation.
- Apply bonding agent evenly, with size small, medium or large, depending on the size of the preparation, along the entire area of an onlay cementation.
- Apply bonding agent to a traditional Class I preparation.
- Treat the entire cavity wall of a Class II restoration in single application.
- Apply bonding agent, with size medium or large, using a wand-like motion to a porcelain veneer preparation.

### Packaging, Article # and Description

Maxi-Brush Easy-Shake Dispenser 150 Series  
150 Applicators, 1 Easy-Shake Dispenser

**935 152**

1 Color,  
Pastel Blue  
Small

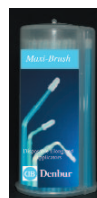

**935 153**

1 Color,  
Teal  
Medium



**935 154**

1 Color,  
Process Blue  
Large

